




ROLLATOR USER MANUAL

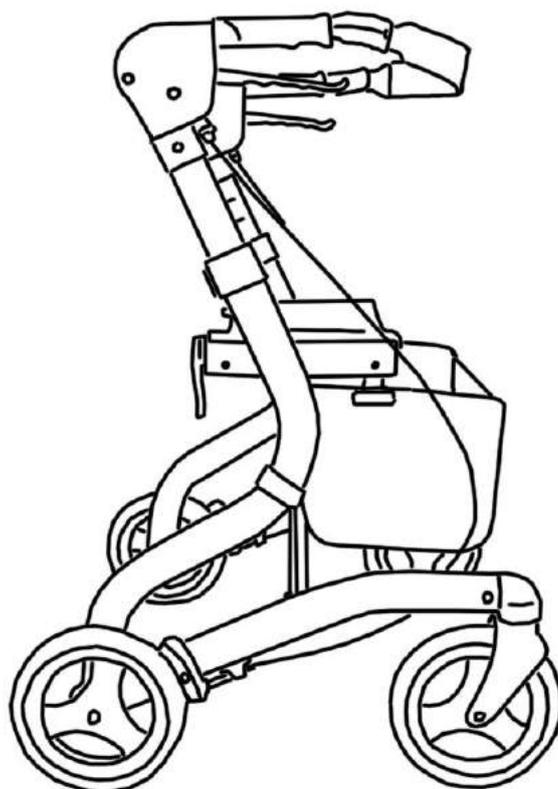

Dear Customer

Thank you for your purchase of our Treka rollator.

Please find the following instructions for use and product specifications.

We wish you all the best with your new rollator.

Product Specifications

The Treka rollator has been designed to assist the user to walk, rest and carry small items of shopping for both indoor and outdoor use on flat surfaces by a single individual with limited mobility with a maximum weight as shown below.

It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the wheeled walker or surrounding property.

SUMMARY

The Treka rollator has been manufactured from high quality materials including the aluminum frame, padded seat and locking brakes. The handles are adjustable in height and the wheeled walker folds easily for transport or storage.


The Treka rollator has been designed to assist the user to walk, rest and carry small items of shopping for both indoor and outdoor use on flat surfaces by a single individual with limited mobility with a maximum weight as shown below.

The Treka rollator has been tested according to BS ENISO 11199-2:2005

Treka Specifications

	Treka 480	Treka 540	Treka 610
User Weight	21.5 st. (136 kg)	21.5 st. (136 kg)	21.5 st. (136 kg)
Total Weight	6.1 kg	6.2 kg	6.3 kg
Seat Width	480 mm (18.9 in.)	480 mm (18.9 in.)	480 mm (18.9 in.)
Seat Depth	200 mm (8 in.)	200 mm (8 in.)	200 mm (8 in.)
Seat Height	480 mm (18.9 in.)	540 mm (21 in.)	610 mm (24 in.)
Handle Height	765 – 855 mm	795 – 885 mm	875 – 995 mm
Overall Width	650 mm (25.5 in.)	650 mm (25.5 in.)	650 mm (25.5 in.)
Folded Width	290 mm (11.5 in.)	290 mm (11.5 in.)	290 mm (11.5 in.)
Wheel	200 mm (8 in.)	200 mm (8 in.)	200 mm (8 in.)

Parts Description

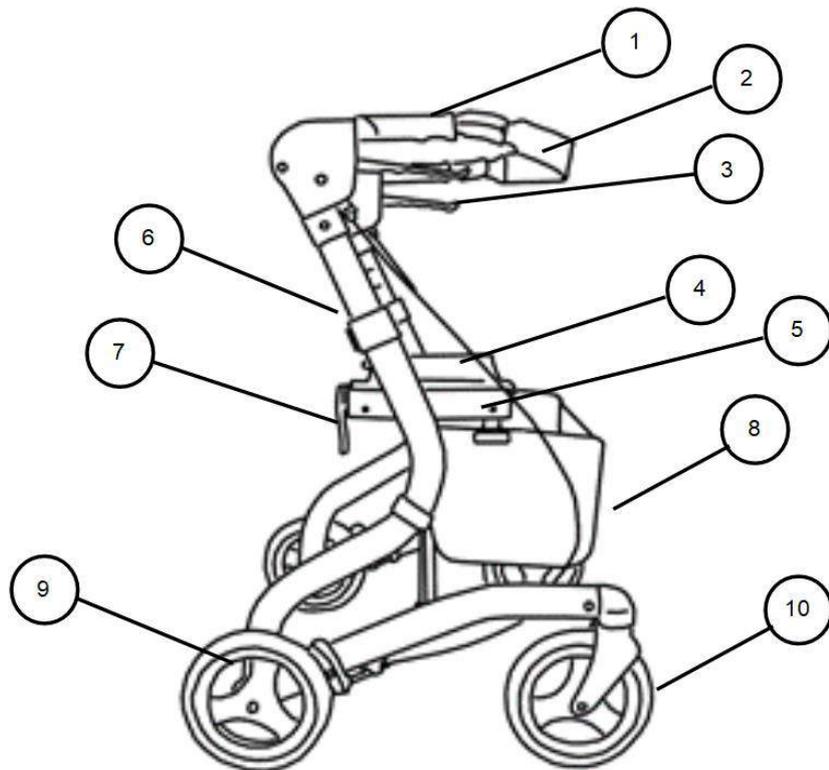

1. Hand Grip	2. Back Rest
3. Brake Lever	4. Pull Strap For Folding
5. Seat	6. Handle Height Adjuster Lever
7. Retaining Clip	8. Rollator Bag
9. Rear Wheel	10. Front Wheel



WARNING!

Ensure that all bolts are securely fastened before using this rollator!
Brakes must be in the parked /locked position before sitting on the rollator!

1. Remove all packaging and set-out parts.
2. **Un-folding the rollator:**
To open the rollator, un-hook the RED retaining clip as shown by lifting upwards.




Clip in locked position.



Lift clip to unlock and unfold rollator.

Push down on either side of the cross bars as shown below.

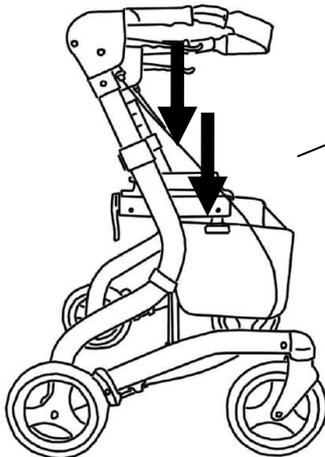

To open rollator push down seat on either side of cross bars.



WARNING!

Be careful not to trap fingers when pushing down seat.

3. Folding the rollator:

To fold the rollator first unlock the RED retaining clip by lifting it upwards as shown below. Then lift the strap in the centre of the seat and continue to lift upwards to fold as shown. Once folded the RED retaining clip can be re-locked to hold the rollator in the folded position for storage etc.



Clip in locked position.




Lift clip upwards to unlock.



Lift strap upwards to fold.

4. Installing the handles:

Holding the 'RED' release lever in the open position as shown below, insert the handles making sure the rectangular slots are facing inwards. Adjust the handles to the required height. Re-adjust the handle height if necessary by releasing the red lever and either lifting or pushing down the handle.

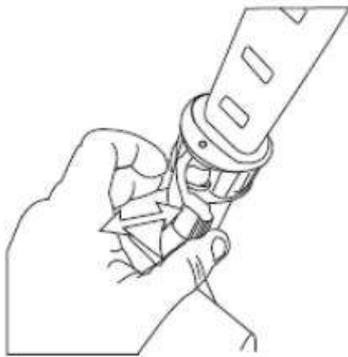

Set to the desired height and close the 'RED' release lever. The release lever should click when locked into the closed position.



WARNING!

Never use the rollator without making sure the RED release lever is fully locked in the closed position.

5. Installing the backrest:




Slide the back strap into the backrest connector making sure the padded side of the back strap is facing to the inside of the rollator.

Secure the back strap into position by tightening the screw and nut.



6. Installing the shopping bag:

Slide the hooks located either side of the bag onto the brackets positioned under the seat at the front.

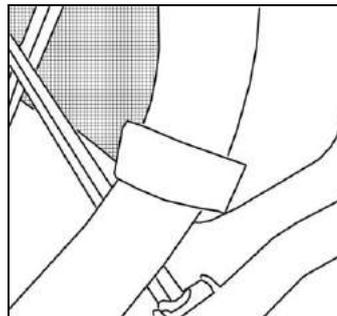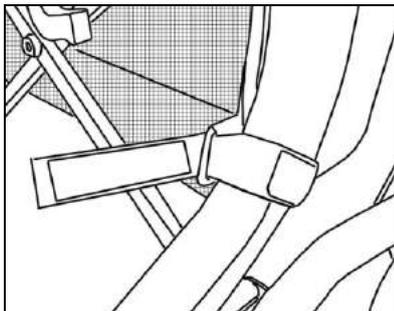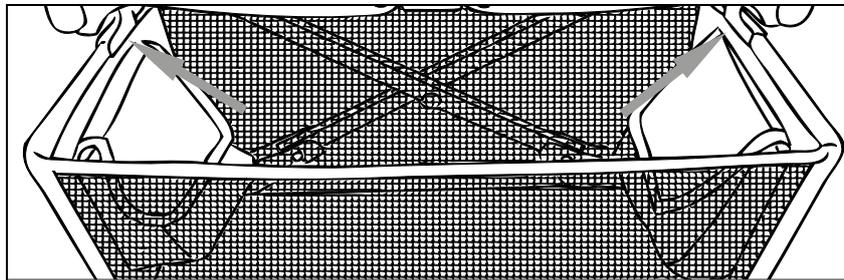

Loop the Velcro strap around the frame and insert through the tab.




WARNING!

Never overload the shopping bag's 5kg Max Load.


7. Braking:

For braking while moving, pull both brake levers upwards as shown and always brake slowly and steadily. Lifting the brake also releases the brake from locked position.

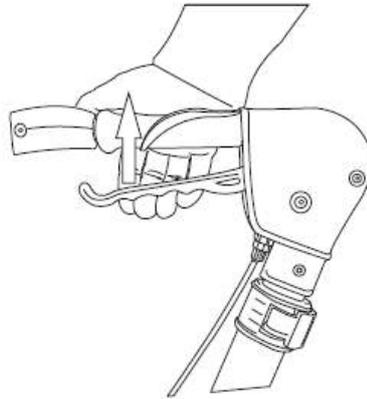

8. Locking the Brakes (Parking Position):

Press both brake levers down to apply and lock the brakes while the rollator is stationary.

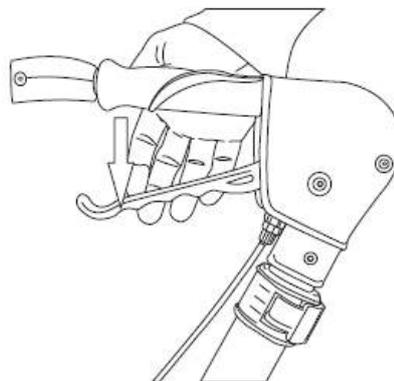


WARNING! Regularly Test The Brakes

9. Sitting on Rollator:

Before sitting lock the brakes into the parking position and make sure the rollator is on an even and firm surface and will not slide or topple over. Sit straight up and never lean backwards.

Do not support your weight on the handle attempting to sit. Use the handles to stabilize your body as you carefully turn around. Keep your hand on the handles and slowly lower your weight onto the seat.

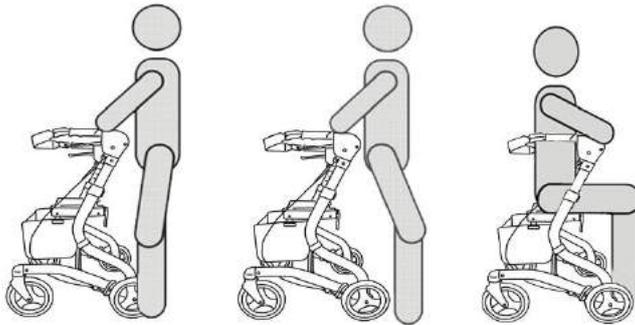

WARNING!

NEVER MOVE ROLLATOR WHILE SEATED



WARNING!

ALWAYS APPLY PARKING BRAKES BEFORE SITTING

Maintenance and other

1. Do not use strong corrosive cleaners to clean or polish surface. This could damage rollator.
2. Store in dry location.
3. Be careful when folding and unfolding rollator to avoid injury by mechanism.
4. Always apply brakes when in stationary position or when seated on rollator.
5. Do not hang bags or other items from handles – this will add stress to the rollator and could also make it un-stable.