

Rollator User Manual

KR *Komfi-Rider*

Dear Customer

Thank you for your purchase of our Zoom rollator.

Please find the following instructions for use and product specifications.

We wish you all the best with your new rollator.

Product Specifications

The Zoom walker has been designed to assist the user to walk, rest and carry small items of shopping for both indoor and outdoor use on flat surfaces by a single individual with limited mobility with a maximum weight as shown below.

It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the wheeled walker or surrounding property.

SUMMARY

The Zoom rollator has been manufactured from high quality materials, aluminum frame, PP, nylon seat, Polyester bag and locking brake. The handles are adjustable in height and the wheeled walker folds easily for transport or storage.

The Zoom rollator has been designed to assist the user to walk, rest and carry small items of shopping for both indoor and outdoor use on flat surfaces by a single individual with limited mobility with a maximum weight as shown below.

The Zoom rollator has been tested according to BS ENISO 11199-2:2005

Zoom Specifications

Overall Length	720 mm (28 in.)
Overall Width	640 mm (25 in.)
Folded Width	270 mm (11 in.)
Handle Height (Max)	880 mm (35 in.)
Handle Height (Min)	780 mm (31 in.)
Seat Width	480 mm (19 in.)
Seat Height	560 mm (22 in.)
Seat Depth	200 mm (8 in.)
User Weight (Max)	21 st. (135 kg)
Total Weight	5.45 kg
Wheels	200 mm (8 in.) Puncture Proof

Parts Description




1. Hand Grip	2. Back Rest
3. Brake Lever	4. Pull Strap For Folding
5. Seat	6. Handle Tightening Knob
7. Rollator Bag	8. Rear Wheel
9. Sick Holder	10. Front Wheel

Assembly & User Instructions

1. Remove all packaging and set-out parts.


2. Inserting Front & Rear Wheels:

Install front and rear wheel sets as shown below.

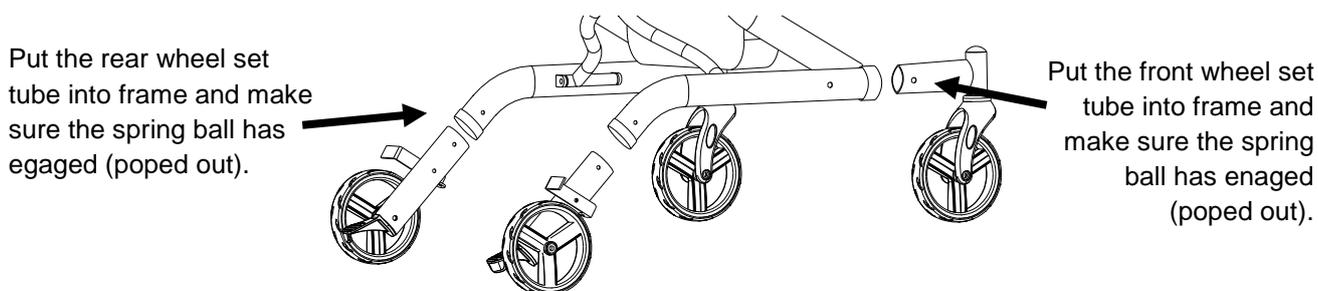

3. Opening Rollator:

To open the rollator, push down on either side of the cross bars as shown below. You should feel both sides 'click' into position.

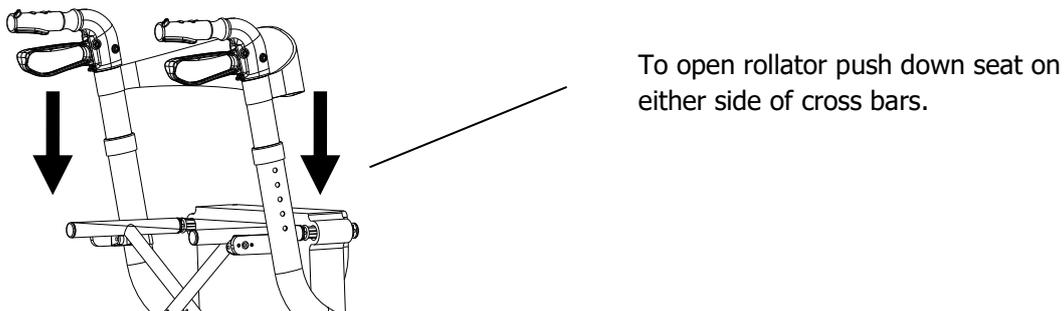


WARNING!

Be careful not to trap fingers when pushing down seat.


4. Inserting the Handles:

Insert the Handles as shown and push down to the required height. Use the tightening knobs provided to secure in place at the desired height.

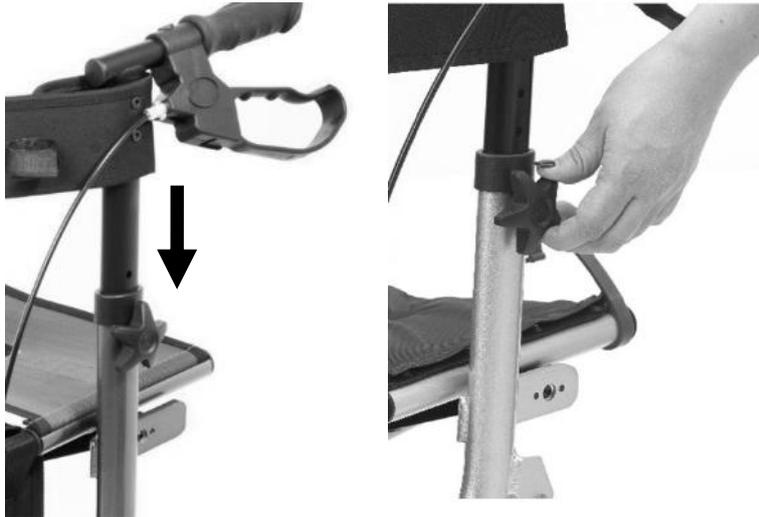

5. Folding the Rollator

To fold the rollator pull up the strap in the middle of the seat as shown below.

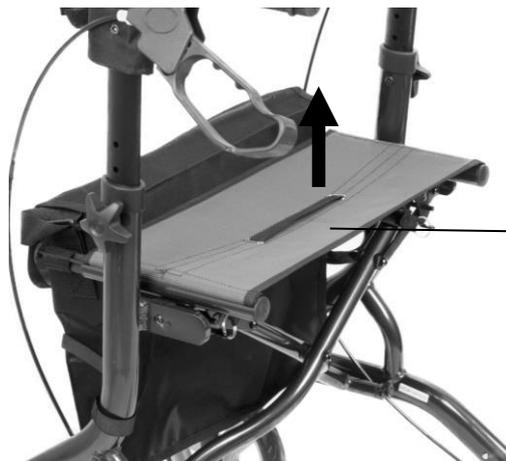


To fold rollator pull up the nylon strap in the center of the seat.

Note: As shown below a band is supplied to hold rollator together in folded position.



6. Rollator Bag:

Attach the bag to the rollator by placing the bag hoops over the two bag hangers positioned at the front of the seat as shown. Place the two velcro bag straps around the rollator frame to secure the lower section of the bag.

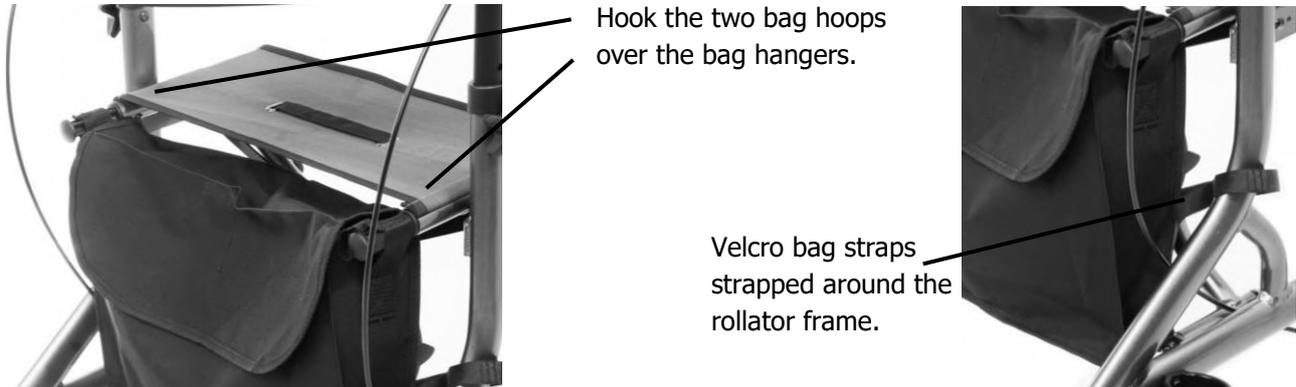

7. Back Strap:

Before using the rollator check that the back strap frame bolts are securely tightened and that the velcro straps are secure.



8. Stick Holder:

Attached to the rollator is a stick holder. Place your stick into the stick holder as shown and secure stick by using the velcro strap attached to the back rest.




9. Handle Height Adjustment:

To adjust the handle height up or down, undo the tightening knob and adjust the height. Tighten the knob back up making sure the handles are secure and level.



10. Braking:

For braking while moving, pull both brake levers upwards as shown and always brake slowly and steadily.

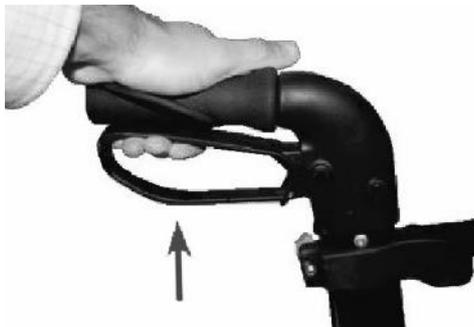

11. Locking the Brakes (Parking Position):

Press both brake levers down to apply and lock the brakes while the rollator is stationary.



WARNING!

Regularly Test The Brakes

12. Sitting on Rollator:

Before sitting lock the brakes into the parking position and make sure the rollator is on an even and firm surface and will not slide or topple over. Sit straight up and never lean backwards.



WARNING! NEVER MOVE ROLLATOR WHILE SEATED



WARNING! ALWAYS APPLY PARKING BRAKES BEFORE SITTING

Maintenance and other

1. Do not use strong corrosive cleaners to clean or polish surface. This could damage rollator.
2. Store in dry location.
3. Be careful when folding and unfolding rollator to avoid injury by mechanism.
4. Always apply brakes when in stationary position or when seated on rollator.
5. Do not hang bags or other items from handles – this will add stress to the rollator and could also make it un-stable.

One Rehab Ltd: info@onerehab.co.uk

KR Komfi-Rider